

Tristel Fuse

Cleaning Solution

(Including **blood** and **body fluids**)



Personal Protective Equipment (PPE) Requirements:



- Wear Apron and gloves at all times.



- Wear Eye protection whilst making up Fuse solution and if there is a risk of splashing when cleaning.

How to Use:



Step 1

Record time, date and initials on container label.



Step 2

Pour 5 litres of cold water into a container.



Step 3

Take one sachet, fold in half and squeeze one side to burst contents through the centre seal. Contents will start to turn yellow.



Step 4

Tear at the serrated edge of the sachet, ensuring it is at arms length.



Step 5

Carefully pour the contents into 5 litres of water.



Step 6

For all all blood or body fluid spillages, wipe up excess with disposable paper towels.
Apply Fuse to surface in an "S" shape with a mop for floors or a cloth for equipment and leave to dry. 5 minutes contact time.



Step 7

After 24 hours dispose of the remaining Fuse solution down the sluice and then flush. Rinse the container with water and clean the outside including the label.



- Make up solution in a well ventilated area.
- Wash hands after removing PPE.
- Store in original packaging in a cool, well ventilated area out of direct sunlight.
- After 24 hours discard any unused solution.

- Dispose of empty packaging into clinical waste.
- Avoid contact with skin and eyes. Contact with the disinfectant may cause mild irritation. Wash affected areas with plenty of soap and water.
- For professional use only.

Tristel order codes:
Fragranced – TSL010101 Unfragranced – TSL010201